

Yslab

2 rue Félix Le Dantec

29000 Quimper

Service Marketing | 2014-02-06

Dimensions : 180 mm * 120mm

Impression Recto/verso

combo insert

Insert EARIGATE combo - AUS - NZ - UK



EARigate™

EARIGATE™ PRODUCTS
The Earigate™ brand offers two unique products:

- **Earigate™**: this unique ear cleaning system cleans and helps prevent ear wax build-up safely. Earigate has developed and patented a proprietary «reverse spray action» delivery system that allows you to safely cleanse your ear canal and helps prevent ear wax build-up. The reverse spray action forces water out of your ear without applying pressure to your sensitive eardrums. Earigate's™ isotonic cleansing solution is 100% natural source sea water, is hypoallergenic and contains no preservatives.

- **Unplugged by Earigate™**: UNPLUGGED Ear Wax Removal System uses Natural Source sea water with a combination of ingredients to safely dissolve ear wax plugs. UNPLUGGED has an innovative «forward spray action» delivery system that sprays along the ear tube to separate the ear wax from the ear canal.

EARigate™



EAR CLEANING SYSTEM


Cleans and helps remove ear wax build-up safely

Earigate™ Ear Cleaning System has a patented «reverse spray action» to safely help prevent ear wax build-up without pressure on your sensitive eardrums. With regular use, the Earigate™ Ear Cleaning System helps prevent ear wax build-up and is 100% natural.




DIRECTIONS


1- Remove over cap. **DO NOT CUT OR REMOVE WHITE TIP**, as this will cause product to spray in the eardrum and may cause ear injury.

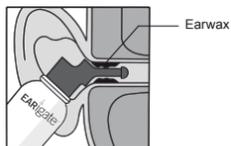

2

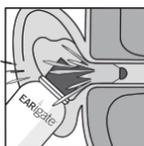

3

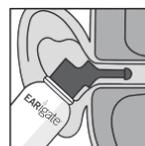

4

2- Over a sink or in the shower, tilting head to the right and insert device into right ear until the cone rests against the opening of the ear

3- Activate: a stream of naturally-sourced seawater is sprayed and washes back out of the ear. The ear is cleaned without any pressure on the eardrum.

4- Tilt head to the left and repeat procedure for the left ear.

5- After use: clean the tip with warm soap water, then rinse.

USAGE

1-2 applications in each ear 2 or 3 times per week. For heavy wax build-up Earigate may be used every day.

Earigate™ Ear Cleaning System is for regular use. In case of impacted ear wax, use the specific product **UNPLUGGED by Earigate™**.

INGREDIENTS

Isotonic 100% natural-source sea water. No preservatives. Hypoallergenic.

WARNING

- Do not use with perforated eardrum or if you have had mastoid surgery. Do not use if you are experiencing ear pain, consult your doctor.

- Product for adults and children. **Not suitable for children under 6 years old.**

- Pressurised container: May burst if heated.

- Keep away from heat/ sparks/ open flames/ hot surfaces.

- No smoking.

- Pressurized container: Do not pierce or burn, even after use.

- Protect from sunlight.

- Do no expose to temperatures exceeding 50°C/122°F.

- Do not freeze.

- Store between 8°C and 35°C.

- Use at room temperature.

- Keep out of reach of children.

Unplugged by EARIGATE



EAR WAX REMOVAL SYSTEM

Helps dissolve ear wax safely


Unplugged by Earigate™

This unique Ear Wax Removal System uses Natural Source sea water with a combination of ingredients to safely dissolve ear wax plugs. UNPLUGGED has an innovative «forward spray action» delivery system that sprays along the ear tube to separate the ear wax from the ear canal.


DIRECTIONS

1- **Shake well before use.**


2- Remove transparent cap. **DO NOT CUT OR REMOVE GREEN TIP**, as this may cause ear injury.

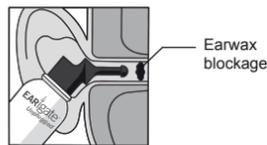

3

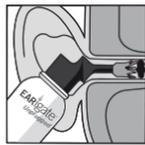

4

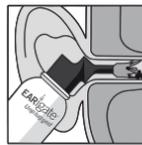

5

3- Gently introduce the tip into the ear.

4- Gently press on the nozzle (for less than 1 second). The UNPLUGGED solution will be directly sprayed along the ear canal.

5- Let the solution act for 5-10 seconds to dissolve the ear wax plug, and spray a second time to rinse.

6- After use: clean the tip with warm soap water, then rinse.

USAGE

In case of ear wax blockage, use 3 times per day for 5 days.

To prevent earwax build-up, use the **EARIGATE™** ear cleaning system 2 or 3 times per week.

INGREDIENTS

Sea water, Water, Glycerin, Apricot kernel oil, Non-ionic surfactant, Niaouli essential oil (Contains Limonen).

WARNING

- Do not use with perforated eardrum or if you have had mastoid surgery. Do not use if you are experiencing ear pain, consult your doctor.

- Product for adults and children. **Not suitable for children under 6 years old.**

- Pressurised container: May burst if heated.

- Keep away from heat/ sparks/ open flames/ hot surfaces.

- No smoking.

- Pressurized container: Do not pierce or burn, even after use.

- Protect from sunlight.

- Do no expose to temperatures exceeding 50°C/122°F.

- Do not freeze.

- Store between 8°C and 35°C.

- Use at room temperature.

- Keep out of reach of children.

EARigate™

DISTRIBUTORS

*Australia: **D M Pharma**

68-72 York St - Sth Melbourne - Vic
3205 Ph 1800 134 111

*New Zealand: **Pharmabroker Sales Ltd**

106 Bush Road - Albany -

Auckland 0632 Ph (09) 415 5050

*UK & Ireland: **Hampton Brands Ltd**

PO Box 419, Twickenham TW1 9GG

Registered Trademark of Earigate, Inc, Burlington, Ontario, Canada.

Made in France
by Yslab - 2 rue Félix Le Dantec
29000 Quimper



NOEAR_CODBLCWV2