



Dry & Irritated Eyes Preservative Free Eye Mist

Please read the instructions in this leaflet carefully before use.

How Boots Dry & Irritated Eyes Preservative Free Eye Mist works

Boots Preservative Free Eye Mist provides instant and soothing relief to dry and irritated eyes. Once sprayed onto closed eyes the mist collects at the edges of the eyelids. Reopening your eye then allows the liposomes in our mist to coat the eye's surface, providing soothing and hydrating relief by restoring moisture to the tear film. This includes relieving:

1. The discomfort of eye dryness and irritation
2. Foreign body sensations
3. The burning and itching of eyes and eyelids.

Eye dryness and irritation can occur due to numerous

environmental and lifestyle factors, such as staring at computer screens, aging, air conditioning and driving long distances.

Eye dryness and irritation is a symptom of the eye's tear film being damaged. The tear film, formed by lipids and fatty acids, provides a protective coating over the surface of the eye, holding in moisture. The stability of the tear film can improve by using Boots Dry & Irritated Eyes

Preservative Free Eye Mist which contains liposomes. By stabilising the tear film, liposomes enhance eye hydration and retain moisture to protect against eye dryness and irritation, frequently associated with contact lens wear.

As well as restoring eye's moisture, this product contains plant extract which provides a soothing effect.

The advanced, preservative free solution is highly tolerated by the eye, allowing users to apply this product frequently throughout the day as required.

Directions for use

When using the bottle for the first time a few sprays are required to fill the pipe to the nozzle.

1. Keeping the eyelids closed, place the spray approximately

- 10 cm from the eye.
2. Spray once or twice for each eye. Use the product 3 or 4 times a day.

Ingredients

Isotonic buffered solution at pH 7.2 containing Phospholipidic complex, extract from Capparis spinosa, extract from Helychrysum italicum, extract from Euphrasia officinalis, extract from Glycyrrhiza glabra.

Important

Do not use the product with make-up which isn't water proof

Keep away from children's reach

Do not use the product after the expiry date printed on the box

Do not swallow

FOR EXTERNAL USE ONLY

Storage

Store at room temperature

Do not freeze

Keep away from heat sources and direct sunlight

KEEP OUT OF SIGHT AND REACH OF CHILDREN



Text revised 01/18

Schalcon S.p.A.
Viale Enrico Ortolani, 195
00125 Rome, Italy

Distributed by
The Boots Company PLC
1 Thane Road Nottingham
England NG2 3AA
www.boots.com

STERILE A

Key to symbols

Read instruction leaflet before use

Store away from direct sunlight

Medical Device Manufacturer as defined in EU directive 93/42/EEC

ARTWORK ONLY

Trident Reference No: **WBAG301263**

Zen Ref: TR1331790

Category: Healthcare

Sub-Category: Eye Care

Brand: HRO

Pack Type: Leaflet

Variants: Dry & Irritated Eyes Preservative Free Eye Mist

Action: C

Date: 31/01/18

Country: UK

Component Code: N/A

Item Code: 80-66-000

CAD Ref No: 145x178mm

Printer: N/A

Substrate: White Paper

Barcode Type: N/A

Barcode Number: N/A

Magnification: N/A

Barcode Truncated By: N/A (smallest bar)

Edgemark Position: N/A

Pharmacode No/NE: N/A

Technical & Non Printing Items

Cutter Guides

Colours

PANTONE 280 C	1



TRIDENT

Connaught House, Connaught Road,
Kingswood Business Park, Hull, HU7 3AP, England.
T: +44 (0) 1482 828100

Please note that any low resolution paper Canon colour copies associated with this job should be referred to for content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS ARTWORK BE ALTERED WITHOUT PRIOR PERMISSION FROM TRIDENT.

STUDIO USE ONLY Sally Platko v1.0

Sm•Art check results: G=1; O=0; R=0; - SP - 31/01/18 13:23:00

TECHNICAL INFO:
PRINTER PLEASE NOTE
Trident have created this artwork to a generic print process specification and repro has not been applied. Please call your Boots contact to discuss any amends or if anything falls beneath your print tolerances.

145 mm

178 mm



ARTWORK ONLY

Trident Reference No: **WBAG301263**

Zen Ref:
Category: Healthcare
Sub-Category: Eye Care
Brand: HRO
Pack Type: Leaflet
Variant:

Action:
Date:
Country: UK

Component Code:
Item Code:

CAD Ref No: 145x178mm
Printer:
Substrate:

Barcode Type:
Barcode Number:
Magnification:
Barcode Truncated By:
(smallest bar)
Edgemark Position:
Pharmacode No/NE:

Technical Et Non Printing Items

 Cutter  Guides



TRIDENT

Connaught House, Connaught Road,
Kingswood Business Park, Hull, HU7 3AP, England.
T: +44 (0) 1482 828100

Please note that any low resolution paper Canon colour copies associated with this job should be referred to for content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS ARTWORK BE ALTERED WITHOUT PRIOR PERMISSION FROM TRIDENT.

STUDIO USE ONLY v1.0
Sm•Art check results:

**TECHNICAL INFO:
PRINTER PLEASE NOTE**
Trident have created this artwork to a generic print process specification and repro has not been applied. Please call your Boots contact to discuss any amends or if anything falls beneath your print tolerances.