



OPTASE® Eye Spray

For dry eyes and eyelids

What is OPTASE® Eye Spray and what is it intended to?

OPTASE® Eye Spray relieves dryness and irritation in the eyes, the eyelids, and the skin around the eyes. Always apply the spray on closed eyelids only. OPTASE® Eye Spray contains sea buckthorn seed oil that slowly spreads on the tear film. The oil strengthens the tear lipid layer to protect the eyes from drying. Also, its natural antioxidant compounds protect the eye surface and the skin against cell damage and speed up their healing. Sodium Hyaluronate supports the water binding capacity of the tear film and lubricates the eyes; it also prevents cell damage and moisturises the surrounding skin. Thanks to its special design the bottle can be used in any position. OPTASE® Eye Spray is a registered Medical Device product with no alcohol, no preservatives, and no ingredients of animal origin.

Instructions for use

Take off the protection cap. Before the first application, test the spraying mechanism a few times on your arm or in the air.

Next hold the bottle approximately 25 cm (10 inches) from the eyes. Close both eyes and then apply 1–2 sprays on the eyelids.


Once you are finished using put the protection cap back on the nozzle head. You may use the spray several times a day and during contact lens wear. If you are using other personal care products or make-up in the eye area, let them settle for at least 10 minutes before applying the spray. Or, use the other product not earlier than 10 minutes after applying the spray. OPTASE® Eye Spray must be discarded 3 months after opening. Expiry date indicated on the bottle.

Precautions and warnings

- Do not use if you are sensitive to any of the ingredients.
- Do not use if the bottle is damaged.
- To avoid contamination, do not touch the eye or skin with the tip of the nozzle.
- Do not touch the tip of the nozzle with fingers.
- If the spray gets directly in the eye, it may cause temporary uncomfortable feelings. If needed, rinse the eyes with clean water. Leave at least 10 minutes between applications of the spray and an ophthalmic medication.
- Keep out of reach and sight of children.
- Store at 5–25 °C, protect from freezing, direct sunshine and heat. Do not use the product in the event it has frozen or it has been stored in hot conditions for a long time.
- Discard the empty bottle appropriately. Due to intentional overfill, a small volume of the solution will remain in the bottle.
- Consult your doctor or pharmacist before you use this product if you are pregnant or breast-feeding.

Composition: Sea buckthorn seed oil, sodium hyaluronate and glycerol in a citrate-borate buffered emulsion.



 Finnsusp Ltd
Pääskykalliontie 5,
FI-21420 Lieto, Finland

 **SCOPE**
Scope Ophthalmics
Unit 2, Amberly Court
County Oak Way
Crawley
RH11 7XL, United Kingdom

Tel UK: 0800 270 0253
Tel Ire: 1800 816 005
E: info@scopeophthalmics.com
www.scopeophthalmics.com



STERILE A

 **0537**

657-1
12/2017
Last revision of text November 2017.



Jaakkoo-Taara Oy
Ruissalontie 4
FI-20200 Turku, Finland
Tel. +358 20 791 7600
E-mail: repro.jt@jt.fi
www.jt.fi

Colours are referential. NOTE! We will charge changes that are against manuscript.

Please add changes as clearly as possible. JT is not responsible of mistakes in approved proof.

JT Job No.: 332286

Font size: 5...12

Item No.: 657-1

Drawn Up: Jari Kontulainen

Colors: Black
diecut

Date: 2017-12-11/07:58:44

Client: PIILOSET BY FINNSUSP OY

Pharmacode:

Dimensions (mm): 70 x 297

Folded (W x L mm): 70 x 27

Drawing:

Material: -

Fonts: ArialMT
Calibri
Calibri-Bold

Approved for printing / Date