Instruction for use

Please read this information carefully because it will help you to use HYCOSAN NIGHT* properly

HYCOSAN NIGHT®

Vitamin A (Retinol palmitate)

For improvement of the tear film and protection of the ocular surface

Preservative Free Vitamin A-containing eye ointment

What is HYCOSAN NIGHT ??

HYCOSAN NIGHT* is a sterile, preservative free, vitamin A-containing eye ointment that contains 250 I.U./g retinol palmitate, liquid paraffin, light liquid paraffin, wool fat and white soft paraffin.

Each tube contains 5 g.

When is HYCOSAN NIGHT* Indicated?

HYCOSAN NIGHT improves the tear film and protects the surface of the eye.

HYCOSAN NIGHT* is a particularly soft and smooth eye ointment which spreads easily on the surface of the eye. The vitamin A contained is a physiological component of the tear film. Thus, the eye ointment is well tolerated and blends well with the existing tear film. The symptoms of burning, tired and dry eyes are relieved and the eyelids move easily over the conjunctiva and cornea.

When should HYCOSAN NIGHT be used?

The symptoms of burning, itching, dry eyes or foreign-body sensation are often experienced due to dry or air conditioned environments. Environmental factors, side effects of medication, age, cigarette smoke, driving at night or prolonged periods working with VDU's can also be the cause of these symptoms.

HYCOSAN NIGHT* relieves these symptoms and forms a comfortable long lasting protective film on the eyes. The greasy consistency of the eye ointment causes a slight impairment of vision. Therefore, HYCOSAN NIGHT* is recommended to be applied before going to bedHYCOSAN NIGHT* prevents any possible difficulty in opening the eyes for the first time in the morning.

How should HYCOSAN NIGHT* be used?

Eye ointments should be used without contacting your eyes or skin with the tube.

Remove the protective cap. The cap is sealed with a white ring, which ensures the security of the tube before first use.

Use HYCOSAN NIGHT* only if the white ring of the tube is intact before first use. Upon first opening the cap is separated from the white ring. Please remove and discard this white ring before using HYCOSAN NIGHT.*



Tilt your head backwards and gently pull on the lower eyelid using a finger of one hand.



With the other hand hold the tube in an upright position above the eye. Instil one strip of eye ointment into the gap formed between the eye and eyelid by gently squeezing on the tube.



Close your eye gently then blink normally.

Replace the cap firmly back onto the tube. To avoid contamination, do not allow the tip of the tube to touch the eye or any other surface.



What further recommendations concerning the application should be followed?

If HYCOSAN NIGHT* is applied during the day vision will be impaired because of the greasy consistency of the eye ointment. You should not drive a car, be near traffic, use machines or work in an unsecure environment during this time.

HYCOSAN NIGHT* must not be used while wearing contact lenses.

HYCOSAN NIGHT* should not be used concomitantly with ophthalmic drugs. If necessary to do so, HYCOSAN NIGHT* should be applied about 30 minutes after the application of the ophthalmic drug.

Please apply the eye ointment by gently pressing on the tube without cracking or rolling up the tube.

There are no restrictions for the use of HYCOSAN NIGHT* in children or pregnant and breast feeding women.

HYCOSAN NIGHT* eye ointment is only for you – it should not be shared with anyone.

How often and how long should HYCOSAN NIGHT* be used?

The frequency of use depends on your personal complaints and habits.

HYCOSAN NIGHT* can be used in addition to lubricating eye drops and is used once a day before going to bed. In cases of severe dry eye sensation HYCOSAN NIGHT* is usually applied three times a day into each eye. If you require to use HYCOSAN NIGHT* more often ask your doctor.

The duration of treatment is not limited.

When should HYCOSAN NIGHT* not be used?

HYCOSAN NIGHT* should not be used when a hypersensitivity to one of its ingredients is known.

HYCOSAN NIGHT* should not be applied while contact lenses are worn.

What do you need to know about storing HYCOSAN NIGHT*?

HYCOSAN NIGHT* eye ointment should not be stored above 25 ℃.

HYCOSAN NIGHT can be used for 6 months after opening.

The shelf life is 3 years.

Do not use beyond the expiry date (imprinted on the packaging carton).

Keep out of the reach of children.

Date of information:

January 2016

Manufacturer ():
URSAPHARM
Industriestraße
D-66129 Saarbrücken
www.ursapharm.de





DISTRIBUTED BY:

Scope Ophthalmics Ltd.

1st Floor, Unit 2 Amberley Court

Whitworth Road, County Oak Way

Crawley, West Sussex, RH11 7XL

Freephone UK: 0800 270 0253,

Freephone IRE: 1800 816005

Fax: +44 203 515 9461

Email: info@scopeophthalmics.com Website: www.scopeophthalmics.com en.